


# How to Collect Oral Fluid


For Therapeutic Drug Monitoring

## Intercept<sup>®</sup> i2<sup>™</sup> Collection Device


To ensure proper collection, check the patient's mouth prior to collecting the sample. If the patient has had anything in his or her mouth within the past 10 minutes, wait 10 minutes prior to collecting the sample.

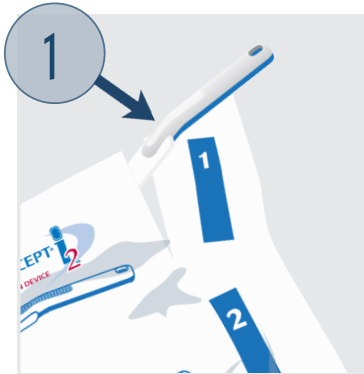


Check the expiration date and remove the collection device pad from the pouch

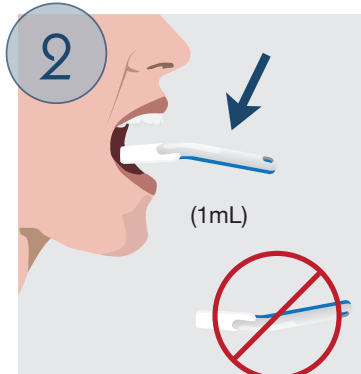


Without touching the pad, insert the pad under the tongue with white side up

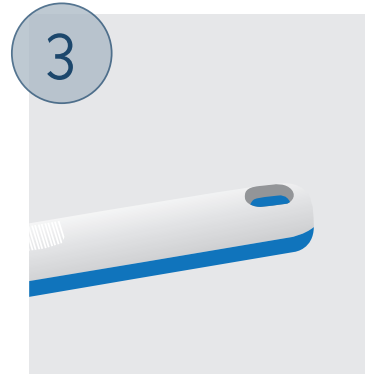


Hold the pad under the tongue until the indicator turns blue, or for 15 minutes at most.

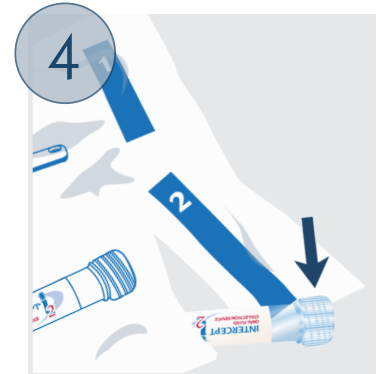


Remove the specimen vial from the collection device package

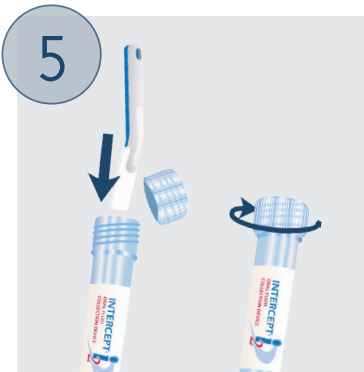


Unscrew the cap and place the collection device pad down in the vial - replace cap tightly

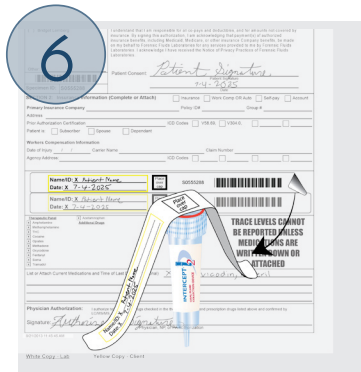

Remove a label from the completed requisition form\* and seal the tube

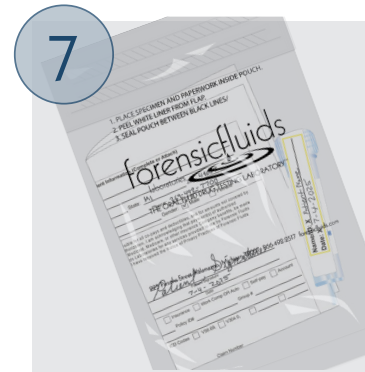

Insert sealed tube and completed form into specimen bag and seal

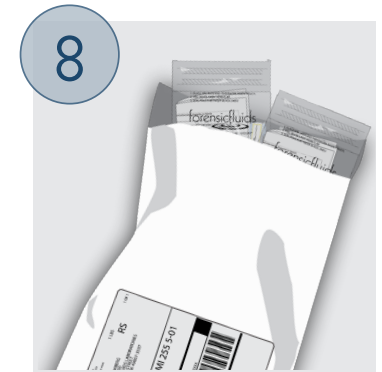

Use the prepaid UPS shipping bags to send samples to FFL

# Questions? Call us at 866.492.2517

forensicfluids<sup>™</sup>  
laboratories