




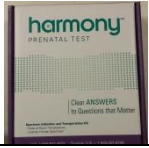

Quick Reference: Sendout Specimen Kit Collection Bronson Laboratory Services

Source	Test	Kit	What to Order?	Kit Requirements	Additional Information
Blood	Harmony Prenatal Test		Sohand	<ul style="list-style-type: none"> Patient must present with kit and completed requisition. Specimen tube in kit. 	Harmony Client Service 1-855-927-4672 Ariosa Diagnostics Inc. San Jose, CA
Blood	Maternit 21		Sohand	<ul style="list-style-type: none"> Patient must present with kit and completed requisition. 	Bronson is the pass- through lab. Results to Physician from Sequemon Lab San Diego, CA (877) 821-7266
Blood	Cystic Carrier Screen or Core Test		Sohand	<ul style="list-style-type: none"> Patient must present with completed Integrated Genetic requisition. Kit is located in S.O. or BWS supply room. Draw two lavenders. 	Bronson is the pass- through lab. Results to Physician from LabCorp Lab Westborough, MA (800) 255-7367
Blood	NxGen		Sohand	<ul style="list-style-type: none"> Patient presents with kit and completed requisition. Special tube in kit. 	Bronson is the pass- through lab. Results to Physician from NxGen MDX Lab Grand Rapids, MI (616) 710-4667 *NxGen courier will pick up*
Blood	Ova 1		Sohand	<ul style="list-style-type: none"> Kit in S.O. Patient must present with completed paperwork. Draw SST-Spin-refrigerate 	Bronson is the pass- through lab. Results to Physician from Aspira Lab Austin, TX (844) 277-4721
Blood	Apifyn		Sohand	<ul style="list-style-type: none"> Kit in S.O. Patient must present with completed requisition. Draw SST- spin-refrigerate 	Bronson is the pass- through lab. Results to Physician from Armune Bioscience Lab, Ann Arbor, MI (844) 427-6863
Blood	Alletess		Sohand	<ul style="list-style-type: none"> Patient must present with kit. Draw SST- spin-refrigerate 	Bronson is the pass- through lab. Results to Physician from Alletess Medical Norwell, MA (800) 225-5404
Blood	Other Kits		Sohand	<ul style="list-style-type: none"> Patient presents with other kit. We will draw- routine processing, and package & send the kit. <p>*No kits requiring dry ice will be sent*</p>	If patient presents with other kits, they must have completed paperwork with billing information. Kit must contain an Airbill for Fedex or UPS.

SOHAND – BMH to draw and send out. Patient must have kit unless these have been preapproved and kits are stored in Sendouts (S.O.). Results are sent to physician only and not BMH Laboratory.

SOMIS – Sendout testing that isn't going to Mayo, and we don't have a Sunquest order code. BMH Laboratory will get results and enter into Sunquest & scan into Epic. Sendouts often knows about these ahead of time and will create a 1x standing order. For follow up Newborn Screen testing (Pompe & Hurler Syndrome MPS1), please contact sendout dept at 341-8488